



TRAZER **XP** Quick Start Guide

Travel System

Before starting, make sure you have all the necessary components and tools.
Initial setup usually takes less than 1 hour to complete.

Components:

- TRAZER **XP** Unit



TRAZER XP UNIT



TRAZER XP POWER CORD

- Accessories



KEYBOARD



REMOTE



HEART RATE MONITOR



HDMI CORD

- TV / Monitor 43"

- Travel System Components



TRAVEL STAND



KEYBOARD TRAY x 2



TV MOUNT
(PRE-INSTALLED ON TV)



TRAVEL CASE



Setup Steps

1 Install the TRAZER XP Unit


- 1.1. Using the TRAZER XP Installation instructions provided, install the unit..

SEE Travel Installation


Proceed to Step 2 once installation and accessory setup are complete.


2 Power On

- 2.1. Turn on both the TRAZER XP Unit and the TV.
2.2. The TRAZER XP Application will load automatically and display on the TV.
2.3. Turn on the Keyboard to navigate the screen - or use the TRAZER Remote.

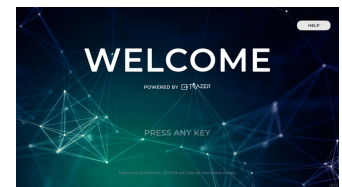

3 Connect to WiFi

- 3.1. Using your Keyboard or TRAZER Remote, click anywhere on the screen to exit the TRAZER XP App.
3.2. To find available networks, click on the  **icon** located in the bottom right of the desktop toolbar.
3.3. Select a network from the list and enter the network password.

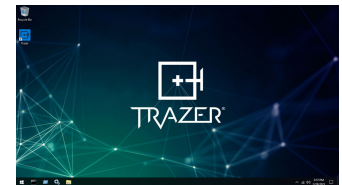


4 Open the TRAZER XP App

- 4.1. Click the  **icon** on the desktop to re-launch the TRAZER XP App.

5 Login

- 5.1. Login with your TRAZER Admin credentials.
(You should have received an email from TRAZER Support (support@trazer.com) regarding your TRAZER account. You will be required to reset your password and verify your email before you can login. If you have not received this email, your Administrator will need to give you Admin access or assist with setup.)

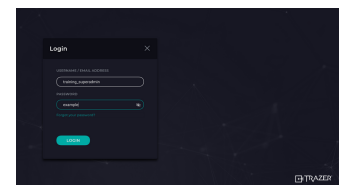


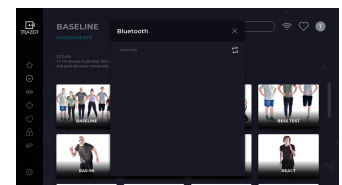
6 Register the TRAZER XP Unit SKU

- 6.1. The TRAZER XP Unit SKU will be located on a sticker on the back of the Unit. Enter the SKU and click **Register**.



7 Connect Heart Rate Monitor

- 7.1. Ensure the Monitor is powered on and being worn correctly.
7.2. Click on the  **icon** in the top right corner of the screen.
7.3. Click **Start Scan** to show available devices.
7.4. Select the Heart Rate Monitor from the list and click **Connect**.
7.5. Once a heart rate is displayed, exit the menu.





TRAZER XP Installation

Travel System Installation

1. Assemble the Avenger stand.

Pull up on the chrome pegs at the bottom of the stand to release the legs. Then pull them down to lock them into place.

2. Mount the TV.

The TV Mount has been pre-installed on the TV.

Loosen the knob on the side of the mount so that the stand pin can fit inside the slot.

Hold the TV above the stand and lower the mount onto the stand pin.

When positioning the TV, ensure one of the stand legs is perpendicular to the front of the TV.

Once positioned, tighten the knob on the side of the mount to secure.

3. Attach the Keyboard Trays.

In alignment with the TV, attach one of the trays 31 inches from the floor for proper unit placement. For the second tray, position at least 8 inches above the first tray. Ensure the trays are tightened as securely as possible.

4. Position the TRAIZER XP Unit and Accessories.

Place the TRAIZER XP Unit on the bottom tray, aligning the front of the unit with the front of the tray.

Place the Keyboard on the top tray.

5. Continue to TRAIZER XP Unit and Accessory Setup.

TRAIZER XP Unit and Accessory Setup

1. Connect Required Cords

a. Plug the TRAIZER XP Unit Power Cord into the accompanying power bank and connect to the left side of the Unit and to a power outlet.

b. Connect the TV Power cord to the TV and to a power outlet.

c. Connect the HDMI Cord from the TV to the left side of the Unit.

d. Plug the TRAIZER Remote wireless receiver into an available USB port on either side of the Unit.

e. Plug the Keyboard wireless receiver into an available USB port on either side of the Unit.

2. Charge Accessories

a. Connect the TRAIZER Remote charger to an available USB port on either side of the Unit and plug the Remote in.

b. Insert batteries into the Keyboard.

c. Connect the Heart Rate Monitor charger to an available USB port on either side of the Unit and plug the Monitor in.

3. Continue to Step 2 of the Setup Steps.